

benefits

-  **EASY USABILITY**
Time saving – gentle dressing change
-  **BIODEGRADEBAL**
Plant based and naturally degradable in the wound
-  **PROTECTION**
Effectively reduces the bacterial load in the wound and removes and prevents biofilm
-  **CONTINUING MOISTURISATION**
Creates and maintains a moist environment and supports autolytic debridement
-  **NATURAL AND TISSUE-FRIENDLY**
Non-sensitizing, non-irritating for wounds, steril



WOUNDGEL

New. Easy and simple to use.



AXXO GmbH
Rödingsmarkt 20
20459 Hamburg
Germany

+49 (0)40 38 919 40
pharma@axxo.de
www.axxo.de

Advanced woundcare




The AXXO Woundgel (which is a generic to the Prontosan® Wound Gel X from B. Braun) is a plant based, non-allergenic hydrogel based on a unique microsphere technology.

It contains PHMB incorporated into fully biodegradable starch microspheres, which are suspended in a gel matrix for optimal infection prevention and treatment. The gel delivers moisture to the wound, whilst maintaining enough absorption capacity for exudate, debris and bacteria. The AXXO Woundgel is a CE-certified medical device of class IIb.


THERAPEUTIC INDICATIONS

Cleansing, moisturizing and decontaminating skin wounds (dry or with low level of exudate):

- Traumatic wounds
- Post-operative wounds
- Chronic skin ulcers (e.g. venous, diabetic or pressure ulcers)
- Thermal wounds
- Radiation-induced wounds

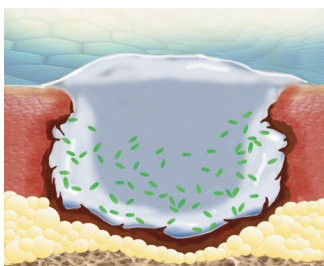

Bacteria and biofilm reduction softens necrotic tissue.

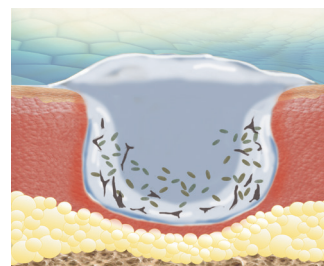

Facilitating autolytic debridement and increase wound healing.



GENERAL USE

AXXO WoundGel should be liberally applied to the wound bed assuring all cavities and pockets are filled. Keep the surface of the wound continuously moist to ensure adequate cleansing and decontamination.

Coatings and necrotic tissue are gently released and removed with the next dressing change.

For best results, the wound and the skin area around the wound should be appropriately cleaned prior to the application of AXXO WoundGel. AXXO WoundGel may remain on the wound between dressing changes and per institutional protocol.

PACKING

Tube (50 g and 100 g)

Primary: Polyfoil®PP-tubes (steam sterilized)

DEHP- and latex-free

Secondary: Folding carton with insert (IFU)

AXXO WoundGel	Volume	PU	Art.-No.	REF
AXXO WoundGel Tube	50 g	1 pc.	440001	960720
AXXO WoundGel Multipack	20x 50 g	20 pcs.	440010	960738
AXXO WoundGel Tube	100 g	1 pc.	440002	960721
AXXO WoundGel Multipack	10x 100 g	10 pcs.	440020	960739

Ingredients: 0.1% PHMB – Polyhexamethylen biguanide (Polyhexanide), Biodegradable Starch Microspheres and Purified Water