Product Catalog

joy [®] **drops**
Lubricants

INDEX CATALOG

COMPANY PROFILE

Since 2000 **Turkuaz Company**, present in Business & Experienced stuff of Administrative and R & D Departments; has aimed at providing consumable Medical goods for all area of application in Hospitals, Dental Clinics, Laboratories, Pharmacy OTC Sectors etc.

In order to accomplish this objective, Turkuaz has kept working hard and making large investments in Research & Development to develop our specific selective formulates with a well organised, disciplined operation task conducted by experts of its management & science teams.

As a consequence of Manufacturing best Quality and specific/selective products with most competitive prices, Turkuaz had to broaden its product range & diverted upon increasing demand of esteemed clients. Apart from such fast growth of the company & its good reputation built in years, we, as Turkuaz Family, are proud of obtaining High Quality Control & Service through Manufacturing and Developing our formulated Products that enable every relevant user to ease their lives.

Quality of our Manufacturing and Products is evidenced by **TS EN ISO 13485:2016 & TS EN ISO 9001:2015, CE, FDA** Certificates, Analysis & Test Chemical and Microbiological Reports, our Manufacturing Process is implemented according to **GMP** & has **International System Standards** to put into practice Quality & Control of all Production Processes from specific and selective Raw Materials (RM) to enantio or diastro selective Finished Product (FP).

Turkuaz Medical Exports to more than 80 countries all over the world including the US, West Europe, North Europe, African, Far Eastern & Middle Eastern Countries. Furthermore, in Domestic Market, Turkuaz is a leading & well-known establishment of its category. Turkuaz covers all product range in terms of sexual health and pleasure field.

- 01 // **Natural** Lubricant Gel
- 02 // **Herbal** Lubricant Gel
- 03 // **Strawberry** Lubricant Gel
- 04 // **Mint** Lubricant Gel
- 05 // **Chocolate** Lubricant Gel
- 06 // **Cherry** Lubricant Gel
- 07 // **Silicone** Lubricant Gel
- 08 // **Warming** Lubricant Gel
- 09 // **Enhancement** Lubricant Gel
- 10 // **Erection** Lubricant Gel
- 11 // **Delay** Lubricant Gel
- 12 // **Delay** Cream
- 13 // **Delay** Spray
- 14 // **Delay** Wet Wipe
- 15 // **Penis** Enlargement Cream
- 16 // **Breast** Firming Cream
- 17 // **Vagina** Tightener Gel
- 18 // **Genital** Bleaching Cream
- 19 // **Intimate For Women**
Hygiene Cleaner Liquid
- 20 // **Intimate For Women**
Hygiene Cleaner Foam
- 21 // **Intimate For Men**
Hygiene Cleaner Liquid
- 22 // **Intimate For Men**
Hygiene Cleaner Foam
- 23 // **Genital Wet Wipe**
Before & After Sex
- 24 // **2in1** Sensual Massage Lubricant Gel
- 25 // Sensual Massage Oil
- 26 // **Sex Toy** Cleaner Spray
- 27 // Production & Quality Certificates

Our Target Is Your Happiness... **For Better Life!**

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Compositions: Deionized water, Hydroxy Ethyl Cellulose, Natural Glycerin, Monopropylene Glycol, Methyl Hydroxybenzoate, Citric Acid

Properties: Natural Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It can be used for vaginal dryness or as lubricant during sexual intercourse. It is compatible with condoms. It does not spoil condom's texture. It is hypo-allergenic, easy to clean and does not stain.

Usage: Simply apply gel on the skin and rub gently before sexual intercourse.

Storage: To be stored inside its box, in a dry and cool environment (ambient temperature between 5-30 °C).

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Compositions: Hydroxy Ethyl Cellulose, Herbal Glycerin, Citric Acid, Ethylhexylglycerin, Phenoxyethanol, Deionized water

Properties: Herbal Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It can be securely used in sexual intercourses due to its natural herbal compounds. Herbal composition of the gel softens and moisturizes the area when it is applied. It can be used for vaginal dryness or as lubricant during sexual intercourse. It is compatible with condoms and does not spoil condom's texture. It is hypo- allergenic, easy to clean and does not stain.

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Compositions: Deionized water, Hydroxy Ethyl Cellulose, Glycerin, MonoPropylene Glycol, Citric Acid, Strawberry Flavour, Sodium Saccharin, Methyl Hydroxybenzoate

Properties: Strawberry Personal Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It cheers you up with its instance and special aroma. It can be used for vaginal dryness or as lubricant during sexual intercourse. Its nutriment-based aromas are compatible with skin's natural pH. It is also compatible with condoms. It does not damage condom's texture. It is hypoallergenic, easy to clean and does not stain.

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Compositions: Hydroxy Ethyl Cellulose, Glycerin, MonoPropylene Glycol, Citric Acid, Mint Flavour, Sodium Saccharin, Methyl Hydroxybenzoate, Deionized water

Properties: Mint Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It adds colourful fantasies and cheers you up by the instance aroma it has. It can be used for vaginal dryness or as lubricant during sexual intercourse. Its nutriment- based aromas are compatible with skin's natural pH. It is also compatible with condoms. It does not damage condom's texture. It is hypoallergenic, easy to clean and does not stain.

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Compositions: Hydroxy Ethyl Cellulose, Glycerin, MonoPropylene Glycol, Citric Acid, Chocolate Flavour, Sodium Saccharin, Methyl Hydroxybenzoate, Deionized water

Properties: Chocolate Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It adds colourful fantasies and cheers you up by the instance aroma it has. It can be used for vaginal dryness or as lubricant during sexual intercourse. Its nutriment-based aromas are compatible with skin's natural pH. It is also compatible with condoms. It does not damage condom's texture. It is hypoallergenic, easy to clean and does not stain.

Usage: Simply apply enough gel to desired area for lubrication.

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Compositions: Hydroxy Ethyl Cellulose, Glycerin, MonoPropylene Glycol, Citric Acid, Cherry Flavour, Sodium Saccharin, Methyl Hydroxybenzoate, Deionized water

Properties: Cherry Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It adds colourful fantasies and cheers you up by the instance aroma it has. It can be used for vaginal dryness or as lubricant during sexual intercourse. Its nutriment- based aromas are compatible with skin's natural pH. It is also compatible with condoms. It does not damage condom's texture. It is hypoallergenic, easy to clean and does not stain.

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Compositions: Dimethicone, Cyclomethicone, Dimethiconol

Properties: Silicone Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. Silicone Lubricant Gel is not absorb by the skin as much as water based lubricants and keeps its lubricating effect moreover, it can be securely and comfortably used under water. It is compatible with condoms. It does not spoil condom's texture. It is hypoallergenic, easy to clean and does not stain.

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Composition: Deionized water, Hydroxy Ethyl Cellulose, Mono Propylene Glycol, Guarana extract, Glycerin, Citric Acid, Propyl Hydroxybenzoate, Methyl Hydroxybenzoate

Properties: It is a topical gel with sensation of warming, it increases the effect cooling or tingling, moisturizing / lubricating. You will feel the warming effect of this product by the first second it is applied. It is compatible with condoms and does not spoil condom's texture. Hypo-allergenic, easy to remove without stain.

Usage: Simply apply enough gel to desired area and rub before sexual intercourse.

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Compositions: Deionized water, Hydroxy Ethyl Cellulose, L-Arginine, Mono Propylene Glycol, Glycerin, Methyl Nicotinate, Citric Acid, Propyl Hydroxybenzoate, Methyl Hydroxybenzoate

Properties: L-Arginine provide higher blood flow to make clitoral and vaginal tissues more sensitive and responsive to sexual stimulation helps to increase the possibility of reaching orgasm. Menthol and L-arginine helps increasing the circulation of the blood and nerves response. Thus, increasing blood flow to the clitoris and stimulating nerve endings. It shows its effect almost immediately when applied onto clitoris. Compatible with skin's natural pH level. It is compatible with condoms and does not spoil condom's texture. Hypo-allergenic, easy to remove without stain.

Usage: Simply apply enough on clitoris and rub gently before sexual activity. If you use regularly you will get better results.

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Compositions: Deionized water, Hydroxy Ethyl Cellulose, L-Arginine, Mono Propylene Glycol, Ginseng Extract, Glycerin, Citric Acid, Propyl Hydroxybenzoate, Methyl Hydroxybenzoate

Properties: L-Arginine provides a higher blood circulation to the genital area by dilating blood vessels and resulting in a better penile erection and hardness as well as an improved orgasm quality. Menthol and L-arginine helps increasing the circulation of the blood and nerves response. Thus, increasing blood flow to the penis with harder erection. A much stronger, longer lasting erection is obtained during this process. It is compatible with condoms and does not spoil condom's texture. Hypo-allergenic, easy to remove without stain.

Usage: Simply apply enough on penis and rub gently before sexual intercourse. If you use regularly you will get better results.

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Compositions: Deionized Water, Caprylic/Capric Triglyceride, Glycerin, Cetearth-25, Disodium Ethylene Dicocamide PEG-15 Disulfate, Acacia Senegal Gum, Behenyl Alcohol, Glyceryl Stearate, Glyceryl Caprylate, Glyceryl Stearate Citrate, Xanthan Gum, Benzyl Alcohol, Potassium Sorbate, Tocopherol, Citric Acid, Undecyl Alcohol, Clove Oil, Glycerin, Olive Oil Peg-7 Esters, Carbomer, Phenoxyethanol, Ethylhexylglycerin

Properties: It helps to delay ejaculation by numbing male organ which reaches to an excessive sensitivity during erection. A pleasant and longlasting sexual intercourse deepens intimacy between couples. It is compatible with condoms and does not spoil condom's texture. Hypo-allergenic, easy to remove without stain.

Usage: Simply apply cream on the hands and rub gently on the penis 5-10 minutes before sexual intercourse for best result.

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Compositions: Ethyl Alcohol, Clove Oil, Mono Propylene Glycol, Glycerin, Citric Acid, Propyl Hydroxybenzoate, Methyl Hydroxybenzoate, Deionized water

Properties: It helps to delay ejaculation by numbing male organ which reaches to an excessive sensitivity during erection. A pleasant and long- lasting sexual intercourse deepens intimacy between couples. It is compatible with condoms and does not spoil condom's texture. Hypo- allergenic, easy to remove without stain.

Usage: Simply apply spray on the penis and rub gently to the foreskin zone, head and underneath your penis, 5-10 minutes before sexual intercourse for best result.

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Properties: It helps to delay ejaculation by numbing male organ which reaches to an excessive sensitivity during erection. A pleasant and long-lasting sexual intercourse deepens intimacy between couples. It is compatible with condoms and does not spoil condom's texture. Hypo- allergenic, easy to remove without stain.

Usage: Simply apply for 1-2 min. by wiping to the foreskin zone, head and underneath your penis, before 5-10 minutes from sexual intercourse for best result.

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Properties: It improves blood circulation to the penis by giving harder erection, thicker penis girth, longer lasting erection, beats premature ejaculation and longer penis length with more pleasurable activity by thickening male organ during sexual intercourse. It is compatible with condoms and does not spoil condom's texture. Hypo- allergenic, easy to remove without stain.

Usage: Apply enough gel to penis 2 times by day. Rub onto your skin using a circular motion and gently 5-10 min before previous to the sexual intercourse. You should use it regularly for better results.

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Compositions: Deionized Water, Caprylic/Capric Triglyceride, Glycerin, Cetearth-25, Disodium Ethylene Dicocamide PEG-15 Disulfate, Acacia Senegal Gum, Behenyl Alcohol, Glyceryl Stearate, Glyceryl Caprylate, Glyceryl Stearate Citrate, Xanthan Gum, Benzyl Alcohol, Potassium Sorbate, Tocopherol, Citric Acid, Undecyl Alcohol, Paraffinum Liquidum, Olive Oil Peg-7 Esters, Commiphora mukul resin extract, Carbomer, Tri Ethanol Amin, Phenoxyethanol, Ethylhexylglycerin

Properties: Breast Firming Cream, which firms the breast shape without a need to go through a plastic surgery, offers a safe use and nurturing functions for breast area.

Usage: Apply enough gel on breast area 2 times by day. Rub into your skin using a circular motion and gently. You should use it regularly for better results.

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Compositions: Hydroxy Ethyl Cellulose, Mono Propylene Glycol, Glycerin, Citric Acid, Chitosan (and) Glycolic Acid (and) Alcohol (and) Glycosaminoglycans (and) Methylparaben, Alüminyum chlorohydrate, Ethyl Paraben, Deionized water

Properties: It is a product that renovates the structure of worn and reduced lips of female organ and enables male and female take more pleasure in sexual intercourse. It provides a sustainable effect in long-term applications and remains effective for about 30-45 minutes after single application. Long-term usage recovers structure of vagina. It is compatible with condoms and does not spoil condom's texture. Hypo- allergenic, easy to remove without stain.

Usage: Apply enough gel on the interior lips of vagina and rub gently before 5-10 min. previous to sexual activity. You should use it regularly for better results.

Storage: To be stored inside it's box, in a dry and cool environment (ambient temperature between 5-30 °C).

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Compositions: Aqua, Caprylic/Capric Triglyceride, Glycerin, Cetareth-25, Disodium Ethylene Dicosamide PEG-15 Disulfate, Acacia Senegal Gum, Behenyl Alcohol, Glyceryl Stearate, Glyceryl Caprylate, Glyceryl Stearate Citrate, Xanthan Gum, Benzyl Alcohol, Potassium Sorbate, Tocopherol, Citric Acid, Undecyl Alcohol, Glycerin, Liquid Paraffin, Olive Oil PEG-7 Esters, Hydrolyzed Wheat Flour, Carbomer, Triethanolamine, Phenoxyethanol, Ethylhexylglycerol

Properties: Reinvent your natural look by Genital Bleaching Cream. It gets rid of the unappealing look of your skin. Dissolves dark spots away and generates a silky layer of skin.

Usage: Apply enough cream by massaging gently 2 times by day to the desired and dark area. You should use it regularly for better results.

Storage: To be stored inside it's box, in a dry and cool environment (ambient temperature between 5-30 °C).

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Intimate For Women Hygiene Cleaner Liquid

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[REF] T0000.319.0001

Compositions: Deionized water, Cocoamido Propyl Betaine, Disodium Cocoamphodiacetate, PEG-200 Hydrogenated Glyceryl Palmate, PEG-7 Glyceryl Cocoate, PEG-120 Methyl Glucose Dioleate, Cocamide DEA, Glycerin, Chamomilla recutita Extract, Fragrance, Citric Acid, Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone, Hexamidine Disethionate

Properties: It helps to remove all dirt and bad odour by killing most of the bacteria and fungal with specially prepared formula. Restores the area through providing a hygienic, pleasant smelling when applied to genital area by a little amount.

Usage: Foam it with water and clean genital area with warm water. For external use only.

Storage: To be stored inside it's box, in a dry and cool environment (ambient temperature between 5-30 °C).

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Our Target Is Your Happiness... For Better Life!

Intimate For Women Hygiene Cleaner Foam

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Compositions: Deionized water, Cocoamido Propyl Betaine, Disodium Cocoamphodiacetate, Chamomilla Recutita Extract, Citric Acid, Fragrance, Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone, Aloe Barbadensis Leaf Juice, Hexamidine Disethionate

Properties: It helps to remove all dirt and bad odour by killing most of the bacteria and fungal with specially prepared formula. Restores the area through providing a hygienic, pleasant smelling when applied to genital area by a little amount.

Usage: Put enough foam to your hand and clean genital area with warm water. For external use only.

Storage: To be stored inside it's box, in a dry and cool environment (ambient temperature between 5-30 °C).

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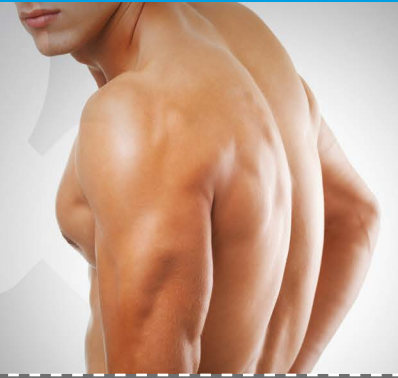
Intimate For Men Hygiene Cleaner Liquid

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Compositions: Deionized water, Cocoamido Propyl Betaine, Disodium Cocoamphodiacetate, PEG-200 Hydrogenated Glyceryl Palmate, PEG-7 Glyceryl Cocoate, PEG-120 Methyl Glucose Dioleate, Cocamide DEA, Glycerin, Chamomilla recutita Extract, Fragrance, Citric Acid, Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone, Hexamidine Disethionate

Properties: Restores the area through providing a hygienic, pleasant smelling when applied to genital area by a little amount.

Usage: Foam it with water and dean genital area with lukewarm water.

Storage: To be stored inside it's box, in a dry and cool environment (ambient temperature between 5-30 °C).

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Our Target Is Your Happiness... For Better Life!

Intimate For Men Hygiene Cleaner Foam

Compositions: Deionized water, Cocoamido Propyl Betaine, Disodium Cocoamphodiacetate, Chamomilla Recutita Extract, Citric Acid, Fragrance, Benzyl Alcohol, Methylchloroisothiazolinone, Methylisothiazolinone, Aloe Barbadensis Leaf Juice, Hexamidine Disethionate

Properties: Restores the area through providing a hygienic, pleasant smelling when applied to genital area by a little amount.

Usage: Put enough foam to your hand and clean genital area with warm water.

Storage: To be stored inside it's box, in a dry and cool environment (ambient temperature between 5-30 °C).

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Compositions: Aqua, Cocoamido Propyl Betaine, Propylene Glycol, PEG 40 Hydrogenated Castor Oil, Fragrance, Allantoin, Lactic Acid, Phenoxyethanol, Ethylhexylglycerint

Properties: It helps to remove all dirt and bad odour by killing most of the bacteria and fungal with specially prepared formula. Restores the genital area through providing a hygienic, pleasant smelling when applied to genital area by wiping.

Usage: Wip the genital area by tissue before or after sexual intercourse.

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Compositions: Hydroxy Ethyl Cellulose, Natural Glycerin, Monopropylene Glycol, Methyl Hydroxybenzoate, Citric Acid, Deionized water
Properties: 2in1 Sensual Massage Lubricant Gel is produced according to EU Medical Devices Directive 93/42/EEC, and CE marking. It can be used for vaginal dryness or as lubricant during sexual intercourse. It can be applied easily to all desired areas to get relaxing effect with its special fragrances. It is compatible with condoms. It does not spoil condom's texture. Hypo- allergenic, easy to clean and does not stain.

Usage: Simple apply Sensual Massage Lubricant Gel to the hand and rub gently to the desired area.

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Compositions: Glycerin, Peg-40 Hydrogenated Castor Oil, Sweet Almond Oil, jojoba Oil, Tocopherol Oil, Fragrance Strawberry

Properties: Moisturizes and smooths the skin. Provides flexibility to your body. Strawberry, apricot and peach flavors are available. Different types of flavors can be prepared depending on your requests at any size.

Usage: Simply apply massage oil to the hand and rub gently to the desired area.

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Compositions: Cocoamido Propyl Betaine, Ethyl Alcohol, Benzyl Alcohol, Methylchloroisothiazolinone (and) Methylisothiazolinone, Deionized water

Properties: It is a Cleaner Spray for cleaning and disinfecting all dirt and bacteria from sex toy and fantasy accessories with absolutely safety and high compatibility with any materials types (Latex, Silicone and Metal).

Usage: It's used as cleaner and disinfectant by spraying directly to all sex toys and fantasy accessories then wash it by water and dry before using.

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