

Remove ticks! Cold.



SAFE AND EASY TICK REMOVAL SET.



The spray **freezes** and **immobilizes** the **tick**



Special **tweezers** enable **easy removal** of the **entire parasite** body



- **Atix spray** is modern preparation that **effectively helps prevent transmission of tick-borne diseases** during parasite removal from the skin.
- **Traditional removal methods may provoke** the tick to **release fluid** from its mouthparts through the skin, potentially releasing pathogenic microorganisms into the bloodstream of humans or animals.
- **Atix spray causes calming** and contraction of the tick's mouthpart walls, **interrupting the flow of fluids** that may contain pathogenic microorganisms.

Instructions for use

- ⊗ **Before application, do not treat** the tick with any substances such as: **Alcohol, Oils, Ointments, Gasoline**
- ❄️ **Application: Spray** the tick **directly**. Point the nozzle **towards the tick's head**. **Press and hold** until one full **dose is released**. Treatment may be repeated **up to three times on the same area** depending on the size of the tick.
- ✋️ **Removal: Grasp** the "frozen" tick with tweezers as **close to the skin as possible**. **Remove in one steady motion**. **Do not squeeze, twist or tear the tick**. After removal, **disinfect both the wound and the instrument**.

