



REUNITE™
Reconstitution made simple



Volume 

Viscosity 

Device Type	Disposable autoinjector
Primary Container	Dual-chamber cartridge Fill volume between 0.5–0.7ml*
Dosage Type	Fixed Dose
Route of Administration	Subcutaneous
Activation Method	Needle cover
Injection Feedback	Audible click at beginning of injection, viewing window

Handling Instructions



1
Unlock the knob to initiate automated reconstitution.



2
Twist off the needle cap to automatically open fluid path and initiate priming.



3
Push the device against the injection site to begin the injection. Wait for the plunger rod to stop moving which indicates end of injection.



+
Discard in sharps container.

Key Features

- ✓ 3-step operation
- ✓ Automatic reconstitution
- ✓ Supports delivery of lyophilized formulations
- ✓ Cap is locked prior to reconstitution
- ✓ Pre-attached Needle Isolation Technology (NIT®)

- ✓ Customizable cannula gauge and length
- ✓ Eliminates manual needle attachment
- ✓ Needle cover activation
- ✓ Permanently hidden needle